## **Finger Splint**

Finger splint is designed for an injury where the thin tendon that straightens the finger is damaged, this is commonly known as 'baseball finger'. This happens when an object hit or has a big impact on the tip of the finger and it bends more than it should.

## Key features:

- 1. Function in stabilizing the finger, control the movement of the joints in its normal range and decrease swelling
- 2. Suitable for finger fracture, sprain, dislocation, straightening mallet finger.
- 3. Made of aluminium and foam for comfort and proper support.
- 4. Built in strap to secure during application.
- 5. Easy to apply and take off.
- 6. Compact and light-weight.
- 7. Easy assessment of blood flow to finger after apply.



## **HOW TO USE:**



Open the splint and position it over the injured finger, with the metal support on top.



Align the splint with the finger and secure the first strap snugly.



Wrap the second strap around for extra stability, ensuring a firm fit.



Check that both straps are secure and the finger is well-supported.

Product name: LAEDI FINGER SPLINT

Brand Name: LAEDI

 $\label{eq:Size} \begin{array}{ll} \text{Size:} & 5.5\text{cm (S)} \ / \ 7.5\text{cm (M)} \ / \ 9.5\text{cm (L)} \\ \text{Basic Packing:} & \text{Each box contain 3pcs including} \end{array}$ 

size S x 1, size M x 1 and size L x 1

Shelf Life: 5 years

Material : Aluminium & foam Colour : Gray aluminium

Properties: Medical Materials & Accessories

Quality Certification: CE/ISO13485

Registered under Act 737 MDA No.: GA9170324-190084 Advertisement approval number: MDAMD 0168/2025

