Malleable Splint Bandage

Malleable Splint Bandage is a simple and mouldable first aid pieces that suggested to be essential during emergency situation. It helps to immobilize injured.

Key features:

- 1. Compact Small size, light-weight
- 2. Application Easy and simple to apply
- 3. Reusable
- 4. Designs Mouldable, unaffected by heat, water and dirt.
- 5. Material Made of soft, malleable strip coated with foam make it rigid after moulding in supporting injured parts.



Use Cases	Immobilize sprained or broken limbs to avoid further
	damage.
Benefits	LIGHTWEIGHT BUT STURDY – The thin aluminum rod will immobilize the limb without weighing down your bag.
Why Choose?	VERSATILE SPLINT: The splint can be shaped, folded, and cut to fit the body as needed, then it can be reused.
Find these products in a First Aid Kit	Limb Injury Pack, First-Aid + Trauma Pack Module – Splint and Airway, or Field Medic Emergency Trauma First-Aid Kit.


STEP 1: MAKE A C-CURVE FOR STRENGTH



Instruction of use:

- 1. Ankle: Splint from foot to mid-calf
- 2. Thumb: Splint from mid-palm to mid-forearm
- 3. Fingers: Splint from the injured fingers to the wrist
- 4. Wrist: Splint from the hand to the elbow

STEP 2: PLACE THE SPLINT ON AFFECTED AREA AND SECURE IT WITH TAPE OR BANDAGE

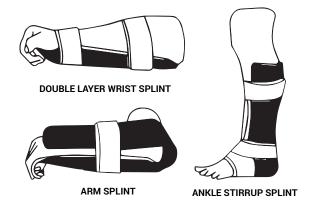

MALLEABLE SPLINT

Product name: LAEDI MALLEABLE SPLINT BANDAGE

Brand Name: LAEDI

Size: 11cm x 90cm

Basic Packing: Individually shrink wrapped

Material: Aluminium & foam

Quality Certification: CE/ISO13485

Registered under Act 737

